Experiences of a total beginner with air abrasion!

I was interested in air abrasion, bumped into Brian at a Cornwall IDP meeting, but was not wishing to spend money. He got me interested, but I didn't want to commit until I had evaluated other systems.

To cut a long story short, I came full circle. Reasons I preferred the Crystal Mark System;

- 1. Quietest I heard in use. Very important not to have loud valve noises and control noises if part of the reason is for apprehensive patients.
- 2. It stops instantly. None of the other systems were as good. Some carried on blowing powder for over five seconds, which is unacceptable to me.
- 3. Engineering quality. Take off the panels and look at any machine. Then look at the Crystal Mark. Some machines are big boxes with nothing in them but a variation on the super cheap machines that most people use for occasional cleaning in the lab. These are no good for serious work on teeth.
- 4. Back up. The kit is personal to Brian, so he responds very quickly to questions. I found him very helpful and available.
- 5. Cuts at **low** pressure with the **least** powder of any machine. I don't want to arrive at the amelo-dentine junction at 160psi, and I don't want to take an age to get there at 40psi using enormous quantities of powder. I wanted the lowest powder, lowest pressure option.

I have had the machine for approx. six months and would say that I use it several times each day. I have just started my first re-fill of powder. What is it good for?

- Brilliant for all fissures, pits, cracks, minor defects of any kind. Nothing better for a confident result. I love the Diagnodent/Crystal Mark combination.
- Can't beat it for salvaging pins in old composites
- Excellent for fast simple cosmetic work on anterior teeth. Uses simply up to the imagination of the operator. Dark lines and cracks just disappear.
- Attrition, abrasion and erosion cases.... multiple teeth at one visit.
- Fast porcelain repairs to the highest possible standard.
- Kids seem to be very calm with this technique. Recent emergency case this last weekend resulted in two new private patients.
- I'm using the machine whenever I want to get the best bond strengths.
- Very useful for cleaning buccal enamel walls in major composite build-ups.
- Surprise revealing of fractures in the floors of heavily restored teeth.
- Cleaning of anything prior to cementing or bonding.
- Can often address a new problem at a scheduled visit while waiting for the patient to anaesthetize for the planned treatment.

I just dashed these notes off today drawing from personal experience so far. This kit grows on you fast once you take the plunge!